NEOVIS® GEL multi

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Stand der Information: 07/2023

EN: FOR SUSTAINED LUBRICATION OF THE EYES

Leaflet

NEOVIS® GEL MULTI
Sodium hyaluronate from fermentation 0.3%.
Hydrogel for sustained lubrication of the eyes.
Sterile, preservative free.

COMPOSITION
1 ml solution contains sodium hyaluronate 3.0 mg, sodium chloride, potassium chloride, disodium phosphate, sodium citrate, magnesium chloride, calcium chloride and water for injections. The solution is hypotonic.

INDICATIONS
Moderate and severe sensations of eye dryness.

CONTRA-INDICATIONS
Hypersensitivity to any constituent of the product.

INTERACTIONS
Do not use NEOVIS® GEL MULTI at the same time as any ophthalmic drugs or products as it may modify their effects.

UNDESIRABLE EFFECTS
In very rare cases transient disorders such as irritation of the conjunctiva, foreign body sensation, redness or burning sensation of the eye, as well as short-term blurred vision may occur.

DOSAGE AND ADMINISTRATION
Remove the tamper-proof ring before the first administration. Remove the protective cap. Tilt the head backwards and position the tip of the dispenser above the eye to be treated. Pull the lower eyelid down with the forefinger of one hand. Squeeze the dispenser and one drop of NEOVIS® GEL MULTI will flow out (see illustrations). If not otherwise recommended, instill 1 or 2 drops of NEOVIS® GEL MULTI into the eye as often as needed. A few blinks will disperse the solution evenly and form a transparent and long-lasting coating on the surface of the eye. NEOVIS® GEL MULTI may also be used while wearing contact lenses (rigid or soft).

PRECAUTIONS
Do not touch the tip of the opened dispenser and do not touch the surface of the eye with the tip of the dispenser. Put the protective cap back on after using NEOVIS® GEL MULTI. Do not use NEOVIS® GEL MULTI if the dispenser is damaged. NEOVIS® GEL MULTI can be used up to 6 months after first use. Any solution not used within 6 months after opening must be discarded. Otherwise the sterility is no longer guaranteed and this may be associated with a risk of infection. Store below 30°C! Do not freeze! Do not use NEOVIS® GEL MULTI after the expiry date indicated on the dispenser and the box. If discomfort persists while using NEOVIS® GEL MULTI, consult a doctor. Keep out of reach of children.

CHARACTERISTICS AND MODE OF ACTION
NEOVIS® GEL MULTI contains sodium hyaluronate, a natural polymer, which is also present in the structures of the human

eye. The particular physical characteristics of sodium hyaluronate confer to NEOVIS® GEL MULTI its viscoelastic and water retaining properties. NEOVIS® GEL MULTI thus provides a stable coating on the surface of the eye which is only slowly eliminated by blinking. Therefore NEOVIS® GEL MULTI combines long lasting relief with maximum comfort. Due to its unique composition NEOVIS® GEL MULTI is very well tolerated. NEOVIS® GEL MULTI is preservative free.

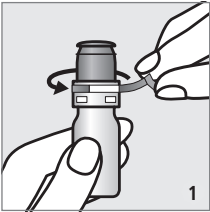
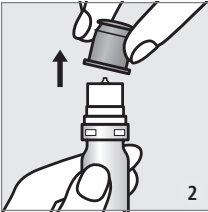
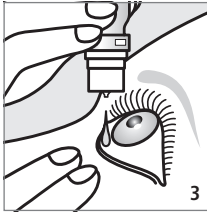
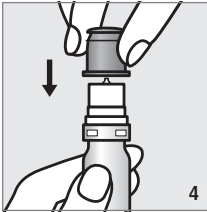
Presentation: 15 ml solution in a multidose dispenser.

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